1. Allgemeine Informationen

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1. General Information

Product description and package contents

This product may only be used with the original system components and in accordance with its product-specific surgical technique and instructions for use.

Information on reuse

Only unused products removed from the original packaging may be implanted. The HALCOR[®] plate and screw are not reusable.

Properties of the material and biocompatibility

This product is made of a wrought titanium-aluminium 6-vanadium 4 alloy in accordance with ISO 5832-3. The entire implant surface has a matt gloss. Osteointegration is not intended.

Warning / precautions

Warnings on the package must always be observed.

This product must only be used in its original condition. Modifications of any kind and mechanical processing are not permitted.

Key to the symbols used on the product label

	Do not reuse
	Do not re-sterilize
	Lot number
	Sterilization by irradiation
	Quantity
	Caution
	Order number
	Expiry date
	Manufacturer

2. Storage and handling

Storage

Sterile implants must always be stored unopened in their protective packaging. The implants are packed in such a way that they can be stored at normal room temperature and relative humidity (corresponding to normal "storage and working environment" in our latitudes; this means about 15°C and 35°C, depending on the time of year, and corresponding normal relative humidity) without risking any impairment of the packaging, the sterility, the product, etc. Do not expose sterile implants to sunlight without protection (i.e. without protection from light by the packaging carton).

Handling

Before using the sterile implant check the package for damage, which may have compromised product sterility. When unpacking the implant, check that it corresponds with the description on the product label (part number and size). When removing the implant from the packaging, appropriate aseptic procedures must be observed by the surgical staff: The outer and, if applicable, the middle PE pouch can be removed by the non-aseptic surgical assistant; the innermost pouch must be removed by the aseptic surgical assistant. The implant must not come into contact with any objects that might damage its surface. Before insertion, every implant must be visually checked for damage.

3. Instructions for Use - Special notes on use

Patient information

Prior to surgery, every patient must be informed in detail about the operation and possible adverse effects. In particular he/she should be advised that and how he/she may influence the safety and longevity of the implant (e.g. by avoiding excessive physical activity and by maintaining normal body weight). The surgeon should record all information given to the patient in writing.

Preoperative planning

Preoperative planning provides essential information on the appropriate prostheses and likely component combinations. Additional implants should be kept at hand in case a different size is required or the selected implant cannot be used.

Surgical technique

Implantation is essentially performed as described in the respective surgical technique. The surgeon must have sufficient training and experience in foot surgery, and must in particular be familiar with this system.

The following items must be observed in particular:

Do not use excessive and jerky force when using the HALCOR[®] plate as a lever to shift bone fragments.

The HALCOR[®] plate must always be tapped in with particularly sensitive, light taps on the hammer. Essentially, the excessive use of force when inserting or fixing the implant must be avoided due to the risk of chip fractures or bone avulsion.

Indications

- Hallux valgus
- Metatarsus primus varus
- Consequences of earlier surgery, e.g. relapsed hallux valgus

Relative indications

- Pseudarthrosis after subcapital metatarsal I osteotomies

Contraindications

- Acute or chronic infections, whether local or systemic.
- Severe muscular, neurological, or vascular disorders affecting the leg in question.
- Lack of bone substance or poor bone quality endangering stable fixation of the plate.
- Any concomitant illness which could compromise functioning of the HALCOR[®] plate.
- Known intolerance of the implant material or individual components

Potential side-effects

The adverse effects listed below are among the most typical and frequent complications of implantation of the HALCOR[®] plate.

Early complications

- Infection
- Haematoma
- Fracture
- Loosening of the implant, esp. the screw fixation
- Breakage of the implant
- Loss of correction
- Venous thrombosis and pulmonary embolism
- Intolerance reactions to the implant material

These side effects should be diagnosed and treated as soon as possible. They do not generally affect the outcome.

Late complications

- Shift of position and loosening
- Breakage of the implant
- Pseudarthrosis
- Relapsed hallux valgus

Note

In the case of confirmed pseudarthrosis, rapid healing or revision with removal of the plate should be aimed for in order to prevent a fatigue fracture of the HALCOR[®] plate.

Interactions with medicinal products

Interactions with medicinal products are unknown.

Postoperative treatment

- In accordance with the surgical technique
- Use recognized procedures and techniques in postoperative care and treatment.
- Document postoperative treatment in accordance with the hospital's standard procedures, e.g.:
 - surgical records
 - postoperative X-rays
 - regular checks / follow-up examinations.

4. Cleaning, disinfection and sterilization/re-sterilization of instruments

The following recommendations are for information only. The manufacturer accepts no liability for the cleaning and sterilizing processes of implants and instruments performed in the purchaser's facilities.

In general the implants are sterilized by means of gamma rays (at least 25 kGy). Implants supplied "sterile" may not be re-sterilized by the buyer.

Instructions for hygiene treatment

The HALCOR[®] instruments are supplied "non-sterile" and must be cleaned, disinfected and sterilized using validated processes before use.

The following instructions are based on validation trials conducted in the course of preparing the medical device for reuse. The person performing the hygiene treatment is responsible for the treatment actually provided achieving the desired results using the equipment and people available in the hygiene treatment facility.

Cleaning and disinfecting by machine

Cleaning and disinfecting by machine should be performed by a process validated by the relevant regulations in a washer-disinfector complying with ÖNORM EN ISO 15883-1 and -2, preferably using a (mild) alkaline cleaning agent. The following program was used for prevalidation:

Phase	H ₂ O-Art	+ H ₂ O (L)	°C	min
Initial rinse	Cold	0	-	2
Wash	Demineralized	+ 3,0	55	7
Neutralize	Hot	0	W	2
Rinse	Hot	0	W	1
Disinfect	Demineralized	0	93	5
Dry	-	-	110	15

Sterilizing

Steam sterilizing with a process validated to ÖNORM EN ISO 17665/ONR 112069 in a steam sterilizer with a fractionated vacuum process as per ÖNORM EN 285. Prevalidation was conducted with the following program specifications:

Cycle type	Temperature	Pressure	Sterilization time	Drying time
Fract. vacuum	134 °C	3050 m bar abs.	5 minutes	25 minutes

Other sterilizing methods/cycles can be used, but these should be validated in accordance with the relevant regulations before being used routinely.

Important

Further information on the use of this product is available from your local sales office.

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